


CareStart™ COVID-19 Antigen Home Test

Rapid test for the detection of antigen of SARS-CoV-2


- For use under an Emergency Use Authorization (EUA) only
- For *in vitro* diagnostic use only




Nasopharyngeal (NP) Swab Collection

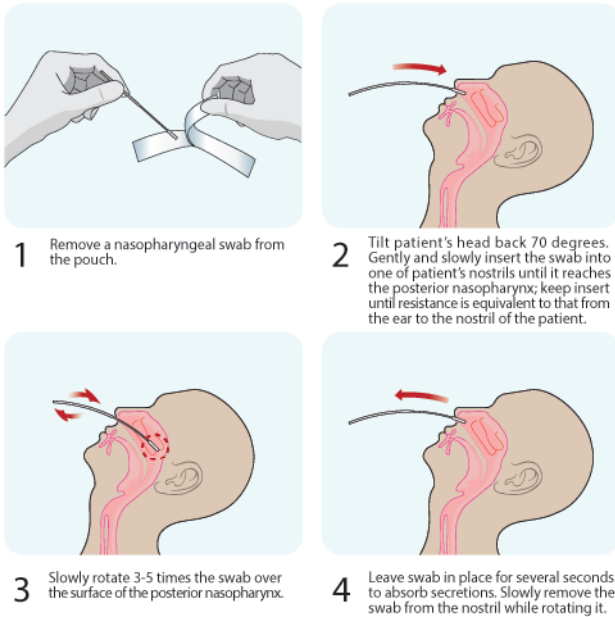

Anterior Nasal Swab Collection

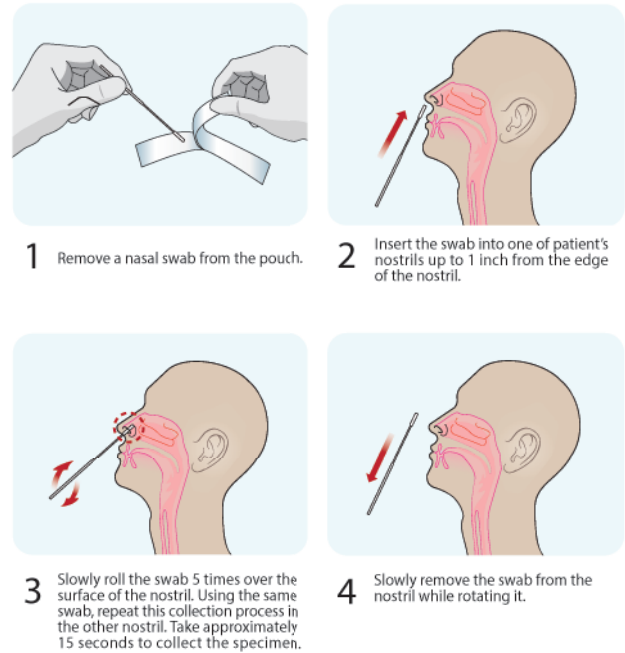

Test Procedures

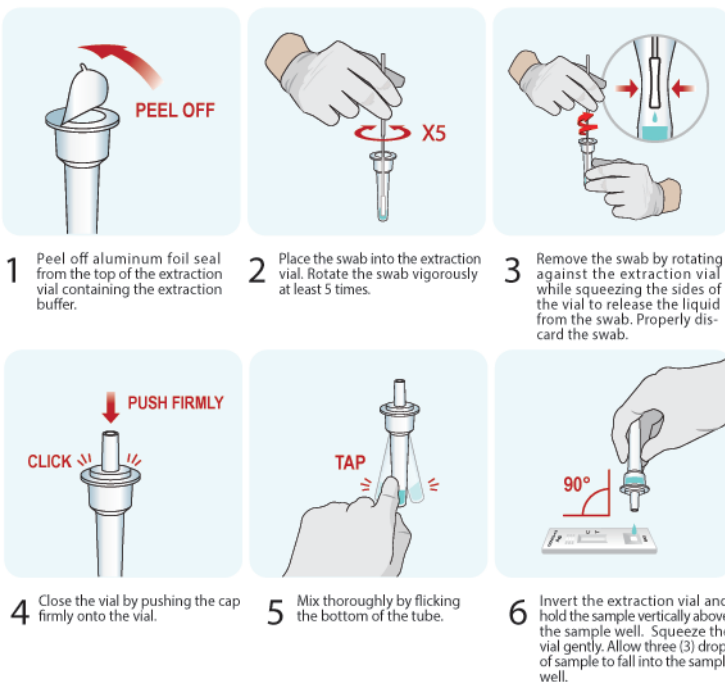

NOTE: Refer to the Package Insert for the cautions.

Start the timer.



Read the result at 10 minutes. The test result should not be read after 15 minutes.



SARS-CoV-2 Positive



Negative



Invalid



NOTES

- The color intensity of the test line varies depending on the amount of SARS-CoV-2 antigen present in the specimen.
- Any faint colored-line in the test region should be considered as positive.

NOTES

- Re-run the test once more using the remaining specimen in the extraction vial.